



**SURFACE
DISINFECTANT**

Lysoformin® spezial

Wipe disinfection of surfaces , floors
and walls



LYSOFORM

CE 0482

Instructions for use - Please read carefully!

Lysoformin® spezial

Concentrate

Wipe Disinfection of Surfaces , Floors and Walls

Composition:

100 g **solution** contains:
2.9 g Polyhexamethylen biguanide · HCl, 9.8 g Didecyldimethylammonium chloride

Range of application:

Disinfection and cleaning

- of surfaces , floors and walls in **one step** only
- for the control of nosocomial infections

Mode of application / dosage:

Lysoformin® spezial is a liquid concentrate with which any use-dilution required can be made by simply adding water (at max. room temperature). Always fill container with tap water first and then add **Lysoformin® spezial**.

Any object/surface coming into contact with food have to be rinsed with drinking water after disinfection/cleaning.

Disinfection and cleaning of surfaces , floors ,walls		
Prevention of nosocomial infections and routine	0.25 %	4 h
	0.50 %	1 h
	0.75 %	30 mn
Virucidal against enveloped viruses* including HBV/HCV/HIV	0,50 %	30 mn
Rota virus	0.50 %	1 mn
Bactericidal against Legionella pneumophila	1.00%	15 mn

* according to recommendation of Robert Koch Institute Berlin/ Federal Health Gazette 01/2004

Lysoform – Hygiene with System

concentration	0,25 %	0,5 %	0,75 %
Dosing System (for 8 l of water)			
Dosing-bag (1 bag = 40 ml)		1 sachet	
Dosing pump (1 stroke = 20 ml)	1 stroke of 20 ml	2 stroke of 20 ml	2 strokes of 30 ml or 3 strokes of 20 ml
Lysoform dosing device (DH 91) no electricity needed.	Adjust once to obtain the desired concentration		

Important: Always pour water (at room temperature) into the container first and then add the disinfectant. The amount of water is calculated by subtracting the amount of disinfectant to be used from the total amount of solution wished for.

Warnings / storage:

- Keep disinfectants in a dry, cool but frost free and well ventilated place and out of reach of children
- Wear suitable protective clothing, adequate protective gloves and eye protection when handling the product.
- Do not use beyond expiry date.
- Shelf-life of the ready-for-use solutions without residue of organic material * and when stored properly (in a cool and dry place) in clean and tightly closed containers: 4 weeks.

“period of time during which the proven efficacy of the ready-for-use disinfectant solution remains unaffected, after preparation at the concentration according to the recommendations of the manufacturer and if it has been stored during the period of time mentioned according to the recommendations of the manufacturer and left to stand unused.”

Properties:

- Control of bacteria (incl. salmonella), fungi, enveloped viruses and rota viruses
- Good compatibility with surfaces and equipments
- Contains no aldehydes, no phenols, no phosphates, no oxidising compounds
- Good cleaning power without addition of detergents
- Fresh, non clinging scent
- Biodegradable

Special precautions:

a) Incompatibilities:

Adding the common detergents to **Lysoformin® spezial** can impair or neutralize its disinfecting properties.

b) Counter-measures in case of improper use:

Should **Lysoformin® spezial** come into contact with the mucous membranes (eyes, throat, stomach etc.) seek medical attention immediately. In all cases, dilute the product without delay: If swallowed, drink **plenty** of water; after contact with the eyes, rinse thoroughly with **plenty** of water.

c) Miscellaneous:

Use disinfectant safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

Standard Packaging/transport units:

200 x 40 ml dosing bags
12 x 1litre bottles
2 x 5 litres canisters
1 x 20 litres canisters

Art.-Nr.

9299

Accessories:

Dosing pump for 5 l canister (1 stroke \cong 20 ml)

Dosing pump for 5 l canister (1 stroke \cong 30 ml)

If you have special questions, call us. We call back immediately.

Lysoform Dr. Hans Rosemann GmbH
Head Office : Jeddah, KSA
mashaly@sapphire.com.sa
mob.: +966 54 287 8749



Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, 12247 Berlin
Telefax 030 - 77 99 2-269, Info.-Telefon 0700LYSOFORM
www.lysoform.de, e-mail: kontakt@lysoform.de



LYSOFORM
Schweizerische Gesellschaft für Antiseptie AG
Postfach 444, 5201 Brugg/Windisch
Telefon 056 - 441 69 81, Telefax 056 - 442 41 14